

Sparkle[™] Prophy Paste Sparkle[™] Free Prophy Paste

REF UPC12/UPCB/UPCBG/UPCCH/UPCM/UPCOV/UPCA/UPMB
UPMBG/UPMCH/UPMM/UPMOV/UPMA/UPFBG/UPSFCC/UPSFCF
UPSFCS/UPSFCWC/UPSFMC/UPSFMF/UPSFMS/UPSFMWC
INSTRUCTIONS FOR USE

Using a prophy angle, fill the cup by pressing the face of the cup into the paste while running the angle at low speed. Keep sufficient paste in the cup to avoid excessive dilution and swallowing of the paste. Saliva evacuation is recommended.

Caution:

For professional use only. Federal Law (USA) prohibits dispensing without a prescription. Keep away from children. Dental prophylaxis should be performed by professionally trained individuals only. Do not swallow. Nausea can result from excessive ingestion of fluoride.







Made in USA

UP-IFU-EN/REVB-08/2021

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted. $©2021\,\text{Hu-Friedy}\,\text{Mfg}$. Co., LLC. All rights reserved.



