

General Purpose Ultrasonic Cleaner Non-Ammoniated Concentrate

REF JEZNA

INSTRUCTIONS FOR USE

Directions For Use:

DO NOT USE ON ALUMINUM.

- Factory sealed for product reliability and protection. To open, place container on a flat steady surface and cut seal slowly. Do not squeeze container.
- THIS PRODUCT IS A CONCENTRATE. Dilute one part solution with ten parts water. Mix one (1) part concentrate with ten (10) parts water. For use directly in an ultrasonic tank.
- Fill tank accordingly per the ultrasonic bath instructions for use. Immerse items to be cleaned directly in tank for 2 to 10 minutes, depending on the volume and amount of debris.
- When cleaning is complete, rinse items thoroughly with water.
- For optimum cleaning results, change solution at least once per day or sooner if solution becomes cloudy or contains a great deal of debris. This helps avoid cross-contamination.

FOR PROFESSIONAL USE ONLY. NOT FOR INTERNAL USE.

DANGER:

Flammable liquid and vapor. Corrosive liquid. Concentrated detergent mixture contains potassium hydroxide and isopropyl alcohol. May cause skin or eye irritation. Harmful if swallowed.

PRECAUTIONS:

Keep away from heat/sparks/open flame/hot surfaces – No smoking. Keep container tightly closed. Take precautionary measures against static discharge. Avoid breathing dust/fumes/gas/mist/vapors/spray. Use with adequate ventilation. Wear protective gloves/protective clothing/ eye protection/face protection. Store in a well-ventilated place, keep cool. Dispose of contents/container in accordance with Local, State, Federal, and Provincial regulations. **IF ON SKIN (or hair):** Remove/take off immediately all contaminated clothing. Rinse skin with water/shower. If skin irritation or rash occurs get medical attention. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. If eye irritation persists get medical attention. **IF ON CLOTHING:** Take off contaminated clothing and wash it before reuse. **IF SWALLOWED:** Rinse mouth. Do NOT induce vomiting. Immediately call a POISON CONTROL CENTER or Physician.

SEE SAFETY DATA SHEET FOR ADDITIONAL DETAILS

24 Hour Emergency Telephone # INFOTRAC: 1.800.535.5053



Crosstex International, Inc.
6789 W. Henrietta Road, Rush, NY 14543 USA
585.359.0130 | 631.582.6777 | crosstex.com

Made in USA
JEZNA / REV C - 03/2022

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.
©2022 Crosstex International, Inc. All rights reserved.

CROSSTEX