

# Safe-Flo™ Adapter

REF M1020ADAP

## INSTRUCTIONS FOR USE

1. Insert the end of the saliva ejector into the blue end of the adapter.



- 2. Insert the valve/saliva ejector combination onto the open end of the high volume evacuation suction line.
- 3. Bend the saliva ejector tube to the desired position and then place into the mouth.
- 4. After single patient use, dispose of the valve/ejector combination in the standard trash.

### CROSSTEX

Crosstex International, Inc.  
 6789 W. Henrietta Road, Rush, NY 14543 USA  
 585.359.0130 | 631.582.6777 | crosstex.com

EC REP MDSS GmbH  
 Schiffgraben 41  
 30175 Hannover, Germany



GH REP STERIS GmbH c/o BDO AG,  
 Längfeldweg 116A,  
 CH-2504 Biel-Bienne,  
 SWITZERLAND



Made in USA  
 M1020ADAP-IFU-EN / REV D - 03/2022

In case of any serious incident that has occurred and involves our devices within the European Union, please report it to Hu-Friedy Mfg. Co., LLC and your national competent authority of the member state.

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.  
 US Pat. No. 8,256,464, 8,714,200, 8,752,579 and foreign counterparts.  
 ©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved.

For professional use only.  
 Not made with natural rubber latex.