
## EN

## Safe-Flo<sup>™</sup> HV Valve

## **REF M1030HVSE**INSTRUCTIONS FOR USE

1. Insert the end of the HVE tip into the larger end of the HV valve.

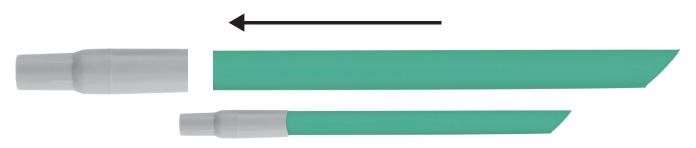

- 2. Insert the valve/HVE tip combination onto the open end of the high-volume evacuation suction line.
- 3. After single patient use, dispose of the valve/HVE tip combination in the standard trash.

## CROSSTEX



Crosstex International, Inc. 6789 W. Henrietta Road, Rush, NY 14543 USA 585.359.0130 | 631.582.6777 | crosstex.com



MDSS GmbH Schiffgraben 41 30175 Hannover, Germany



CH\_REP STERIS GmbH c/o BDO AG, Längfeldweg 116A, CH-2504 Biel-Bienne, SWITZERLAND









For professional use only.

Not made with natural rubber latex.

Made in USA

M1030HVSE-IFU-EN / REV D - 03/2022

In case of any serious incident that has occurred and involves our devices within the European Union, please report it to Hu-Friedy Mfg. Co., LLC and your national competent authority of the member state.

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted. US Pat. No. 8,256,464, 8,714,200, 8,752,579 and foreign counterparts.

©2022 Hu-Friedy Mfg. Co., LLC. All rights reserved.

