HVE Mirror Connect

High Volume Evacuation

REF HVEMC

INSTRUCTIONS FOR USE

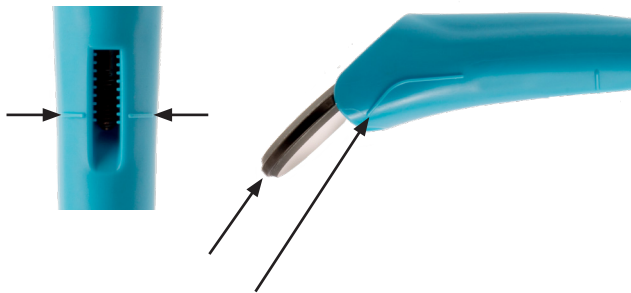
The HVE Mirror Connect is a single use item intended for connection with a suction system for use during dental procedures to remove blood, saliva, bone, tissue, and dental products as well as provide cheek retraction. The HVE Mirror Connect is designed to integrate only with a HuFriedyGroup doublesided #5 mirror head.

Warning:

- HVE Mirror Connect is a single-use product and is not intended to be autoclaved.
- Autoclave the Mirror Head component separately after every use. Refer to the instructions for use provided for mirror head(s), such as the HuFriedyGroup Instrument Reprocessing Guide for HuFriedyGroup Mirror Heads.
- Will not work with simple stem mirror head.

HVE Mirror Connect Instructions for use:

1. Insert HVE Mirror Connect into your HVE valve.
2. Using embossed guidelines on the HVE, slide HuFriedyGroup #5 Doublesided Mirror Head into HVE Mirror Connect fully until Mirror Head locks into place to match orientation.

3. Use HVE Mirror Connect on patient during procedure.
4. After procedure is completed, remove Mirror Head from HVE Mirror Connect.
5. Dispose of HVE Mirror Connect.

Note: The HVE Mirror Connect includes embossed guidelines on the HVE to aid Mirror Head insertion. Two curved guidelines on the HVE sides designate the orientation of the Mirror Head, and one straight guideline on the shaft indicates the appropriate depth of Mirror Head insertion.

HuFriedyGroup #5 Mouth Mirrors	
MIR5DSHDB	#5 HD Black Line Double-Sided Mirror
MIR5DSHD	#5 HD Double-Sided Mirror
MIR5DS	#5 Rhodium Double-Sided Mirror

Hu-Friedy Mfg. Co., LLC
 3232 N. Rockwell Street | Chicago, IL 60618 | USA
 1-800-Hu-Friedy | info@hu-friedy.com

HFL-738 / REV B - 10/2023

All company and product names are trademarks of Hu-Friedy Mfg. Co., LLC, its affiliates or related companies, unless otherwise noted.
 ©2023 Hu-Friedy Mfg. Co., LLC. All rights reserved.

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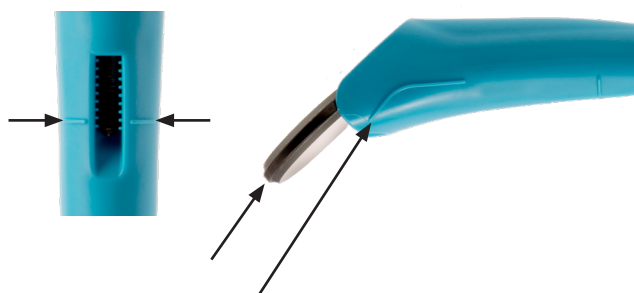
HuFriedyGroup #5 Mouth Mirrors

MIR5DSHDB	#5 HD Black Line Double-Sided Mirror
MIR5DSHD	#5 HD Double-Sided Mirror
MIR5DS	#5 Rhodium Double-Sided Mirror

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©2023 Hu-Friedy Mfg. Co., LLC. Tous droits réservés.