



Contains Six

IMS® Needle Cap Holder

Part Code: IM1003
Stainless Steel

STERILIZE BEFORE USE.

CAUTION: Overextending, bending or attempting to modify can cause breakage. Needle Cap Holder is recommended for use with IMS Infinity and Signature Series Cassettes only.

Steam sterilize for at least 4 minutes at 270°F/132°C or 30 minutes at 250°F/121°C. Do not exceed 350°F/177°C. Recommend 30-minute dry time after sterilization cycle.

INSTRUCTIONS

Infinity Series: Place on base of cassette. Avoid metal flanges of the cassette. Slide through the round infinity symbol. Push against the edge until it locks into place.

Signature Series: Place in a corner on the base of cassette. Slide through the holes on the bottom of the base. Push against the edge until it locks into place.

For How-To Videos, visit
www.Hu-Friedy.com/IMSMaintenance

Manufactured for: Hu-Friedy Mfg. Co., LLC
3232 N. Rockwell St., Chicago, IL 60618 USA
1-800-Hu-Friedy | Hu-Friedy.com

Hu-Friedy Mfg. Co., LLC
Zweigniederlassung Deutschland
Kleines Oschle 8 | D-78532 Tuttlingen

MADE IN USA B2944/0618

