



Contains Six

IMS® Cassette Clip and Needle Cap Holder

Part Code: IM1004
Stainless Steel


STERILIZE BEFORE USE.

CAUTION: Overextending, bending or attempting to modify can cause breakage. Needle Cap Holder is recommended for use with IMS Infinity and Signature Series Cassettes only.

Steam sterilize for at least 4 minutes at 270°F/132°C or 30 minutes at 250°F/121°C. Do not exceed 350°F/177°C. Recommend 30-minute dry time after sterilization cycle.

INSTRUCTIONS

Use clip on lid of cassette.

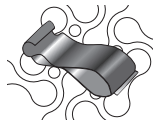

For How-To Videos, visit
www.Hu-Friedy.com/IMSMaintenance

Manufactured for:
Hu-Friedy Mfg. Co., LLC
3232 N. Rockwell St., Chicago, IL 60618 USA
1-800-Hu-Friedy | Hu-Friedy.com
Hu-Friedy Mfg. Co., LLC
Zweigniederlassung Deutschland
Kleines Öschle 8 | D-78532 Tuttlingen

MADE IN USA B2944/0618

Place Needle
Cap Holder
on base of
cassette.

