Pantone 326C - IMS Green
Pantone 1945C - HF Red
Pantone 425C
Black

9.5"

DIRECTIONS

For professional use only. Use with water at temperature between 90° and 140° F (32° to 60° C), demineralized water is ideal. **Presoak:** Add 1 packet per 1 gallon (3.8L) of water. Soak for 3-5 minutes. Rinse thoroughly. **Linen Presoak:** Add 1 to 2 packets per 1 gallon of water (3.8L) per 1 gallon (3.8L) of water. Soak clothing for 10 minutes. Proceed with normal washing. **Ultrasonic Cleaning:** Add 1 packet per 1 gallon (3.8L) of water. Rinse instruments thoroughly, dry and proceed with sterilization. **Evacuators:** Add 2 packets per 1 gallon (3.8L) of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly. Change solution daily or when visibly soiled.

DANGER

Causes skin irritation. Causes serious eye damage. May cause allergy or asthma symptoms or breathing difficulties if inhaled.

Avoid breathing dust/fume/gas/mist/vapours/spray. Wash hands, forearms and face thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. In case of inadequate ventilation wear respiratory protection. **IF ON SKIN:** Wash with plenty of water. Take off contaminated clothing and wash it before reuse. If skin irritation occurs: Get medical advice/attention. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: Call a POISON CENTER/doctor. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Dispose of contents/container in accordance with local, regional, national and/or international regulations.

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