# STERILIZATION STEAM INDICATORS STEAM

Steam Sterilization Indicators used to audit the time-temperature-moisture process necessary for sterilization.

Note: Sterilization indicators are intended for use as an adjunct for assessing the effectiveness of sterilization processes.

For use in steam sterilizers 250°F–285°F (121°C–141°C)

Processed and unprocessed strips should be stored at room temperature between 59°F–86°F (15°C–30°C) away from sources of steam and moisture.

## Directions for use

1. Place indicator strip in the center of each package to be sterilized.
2. After steam sterilization, when pack is opened for use, examine the strip.
3. Both left and right indicator bars must be as dark as, or darker than, center color standard. If left or right indicator is NOT as dark as, or darker than, center color standard, inadequate processing has occurred, and the package must not be used.
4. To ensure proper processing, do not place device directly on the strip.
5. To document use of properly processed instruments, record patient name and date on reverse side of strip.
6. Minimum time and temperature required for bars to change to dark: 12 minutes at 250°F (121°C).

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Hu-Friedy Mfg. Co., LLC