



Contains: 6 (Six)

## IMS™ Needle Cap Holder

Part Code: IM1003

Stainless Steel

### STERILIZE BEFORE USE.

CAUTION: Overextending, bending or attempting to modify can cause breakage. Needle Cap Holder is recommended for use with IMS Infinity and Signature Series Cassettes only.



[www.hu-friedy.com/Reprocessing/](http://www.hu-friedy.com/Reprocessing/)  
1-800-Hu-Friedy; [info@hu-friedy.com](mailto:info@hu-friedy.com)



134°C  
Sterilizable in steam sterilizer  
at 134°C / 273°F for 4 min  
Washer-disinfector for thermal  
disinfection possible


### INSTRUCTIONS

**Infinity Series:** Place on base of cassette. Avoid metal flanges of the cassette. Slide through the round infinity symbol. Push against the edge until it locks into place.


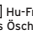
**Signature Series:** Place in a corner on the base of cassette. Slide through the holes on the bottom of the base. Push against the edge until it locks into place.

For How-To Videos, visit

[www.Hu-Friedy.com/IMSMaintenance](http://www.Hu-Friedy.com/IMSMaintenance)

 Hu-Friedy Mfg. Co., LLC  
3232 N. Rockwell St.  
Chicago, IL 60618 USA

MADE IN USA

  Hu-Friedy Europe LLC & Co. KG  
Kleines Oschle 8  
78532 Tuttlingen  
Germany



82944/0820

