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Hu-Friedy Nitrile Flock-lined Utility Gloves

Hu-Friedy brand nitrile flock-lined gloves meet the requirements of the European Directive 89/686/EEC for personal protective equipment, as well as the European standards EN420, EN374, and EN388 for protective gloves. Specific information relative to these standards and directive is as follows:

RISK CLASS: 3 LENGTH: 330 mm THICKNESS: 0.38mm

PERFORMANCE LEVEL per EN388:4101¹
MICRO-ORGANISM PROTECTION, per EN374-2: SI
CHEMICAL RESISTANCE PERFORMANCE LEVELS, per EN374-3

CHEMICAL PRODUCTS	PENETRATION TIME (in minutes) ²	PERMEATION INDEX (1 to 6) ²
Methanol	40	2
n-heptane	>480	6
Sodium Hydroxide 40%	>480	6
Sulphuric Acid 96%	106	3

¹ **Mechanical Risks:** The numbers refer to, in order, the levels of resistance to Abrasion (0 to 4), Cut (0 to 5), Tear (0 to 4) and Puncture (0 to 4). The higher the index number, the more resistant the glove is to the indicated challenge.

² **Penetration Time and Permeation Index:** Indicates the time needed for the chemical to penetrate into the glove. The higher the index number, the greater the permeation resistance.

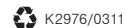
User instructions and recommendations:

- 1.) Do not use if sensitive to dithiocarbamates.
- 2.) Avoid contact with ketons and nitrogenous organic compounds.
- 3.) Prior to use immerse glove in chemicals as a preliminary trial.
- 4.) Check physical appearance of gloves; if swelling or other physical change is noted substitute with a new pair.
- 5.) Turn cuff of glove down to avoid drainage of chemicals into glove.
- 6.) After use decontaminate gloves prior to taking them off.
 - When using solvents, oils and chemicals remove excess solvent and wipe off with a dry cloth.
 - When using acids, alkalis and detergents rinse the gloves with water; then wipe off gloves with a dry cloth.
- 7.) Place gloves into separate sterilizer pouch.
- 8.) Steam autoclave up to 250°F/121°C.
- 9.) Ideal storage conditions 59°F/15°C to 86°F/30°C, 40% to 60% humidity and protected from sunlight.

Note: Glove performance and instructions are based on laboratory testing data. User results may vary based on individual condition of use. Gloves can be steam autoclaved up to five (5) times.

Nitrile flock-lined gloves are available in the following sizes:

Part Code (REF)	Glove Size
40-060	7
40-062	8
40-064	9
40-066	10

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