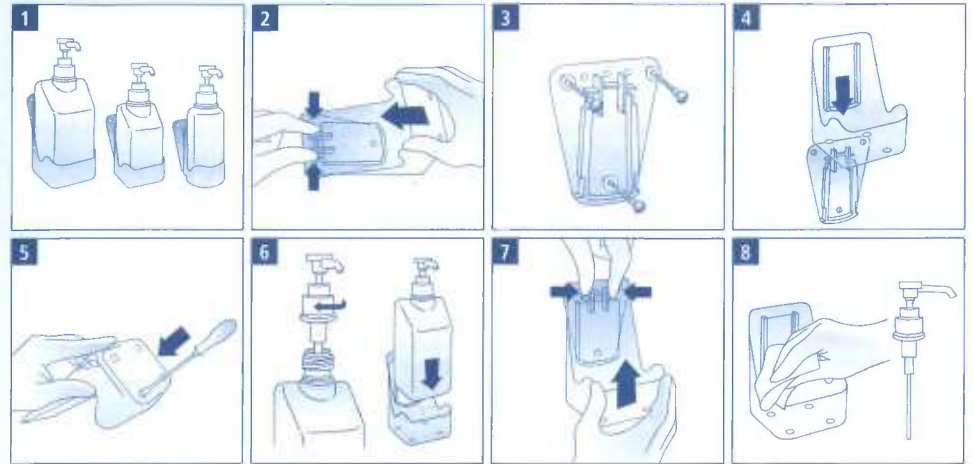
BODE Wandhalter

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BODE Wandhalter

MONTAGE

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EN INSTRUCTIONS FOR USE**BODE Wandhalter****MOUNTING**

- To mount the wall-mounted holder, first dismantle the wall plate and the shell. To do this, lightly push together the two lugs at the top of the wall-plate and push the shell over the bracing fixture (Fig. 2).
- Insert the wall plugs supplied into the drill holes made in the wall, and fasten down the wall-mounted plate, with the plain washers and screws also supplied (Fig. 3).
- Once the wall plate is attached, push the shell onto the wall plate from above, until the bracing fixture engages. The wall-mounted holder is now ready for use (Fig. 4).
- If the wall-mounted holder is used to hold wash or shower lotions in humid areas, it is advisable to penetrate the holes provided on the underside of the shell with a screwdriver. This will allow any water that collects to drain away (Fig. 5).

OPERATION

- Screw the pump onto the bottle and place both of them into the shell of the wall-mounted holder. The bottle nozzle sits inside the shell and cannot slip. It can also be elbow-operated (Fig. 6).

CARE AND MAINTENANCE

- The shell can be cleaned or replaced easily by pushing together the two lugs at the top of the wall-mounted plate and by pushing the shell upwards over the bracing fixture (Fig. 7).
- The inside and outside surfaces of the wall-mounted holder can be disinfected if necessary, otherwise wipe a cleaning agent. Clean the pump whenever necessary, but at least every 2 months. Do not use extra strong cleaning agents (Fig. 8).

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BODE Chemie GmbH