EN On purchase of these disposable paper filters you will receive a product of high quality, compatible to Hu-Friedy sterilization containers.

1. IMCO-FIL3M and Description:

Disposable paper filter without indicator for Hu-Friedy Dental Sterilization Containers (100 pieces/pack)

2. Handling

Place one sheet of the appropriate paper filter over each perforated section on the inside of the container lid and bottom. Once the Filter is in place, secure the filter with the retention plate.

Creped Sterilization Paper Filters allow the penetration of saturated steam during steam sterilization process.

When the filters are used with Hu-Friedy Sterilization containers, the Container system is able to maintain sterility of the sterilized load for a maximum period of 30 days.

3. Storage

The product shall be stored in its original packaging in dry conditions (35-50% relative humidity, 59-86°F (15-30°C))

4. Specific Details

The following information should carefully be considered by the user in order to prevent undesired malfunction of the filters. If any of the listed actions occur, the filter must be replaced:

- Never puncture the filter
- Avoid leaving pointed and sharp objects on the sterilization container
- Avoid spilling of any kind of fluid (cleaning or disinfection solutions) on the container
- Avoid rubbing dust and dirt into the filter when wiping off the container
- Avoid long-term storage

5. Disposal

After use dispose the filter in accordance with regulations of your waste disposal.

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- Keep Dry
- Charge number, identification assigned to a particular quantity or lot of material from the manufacturer
- Read the instruction for use
- Manufacturer's declaration that the products comply with the essential requirements of the European Medical Devices Directives
- Use-by Date
- Single use only

**DISPOSABLE PAPER FILTERS, WITHOUT INDICATOR
 FOR 'M' VERSION CONTAINERS - 100 PIECES
 INSTRUCTIONS FOR USE**

| IMCO-FIL3M

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Hu-Friedy Mfg. Co., LLC: 3232 N. Rockwell St. | Chicago, IL 60618 | USA
Hu-Friedy Mfg. Co., LLC: Zweigniederlassung Deutschland