SWERV™

MAGNETOSTRICTIVE SCALER HANDPIECE

User Manual • Benutzerhandbuch • Manuel de l'utilisateur • Manuale per l'utente
Manual del Usuario • Manual do Usuário • ユーザーマニュアル

Hu-Friedy®

SWERV™

MAGNETOSTRICTIVE SCALER

USER MANUAL

INTRODUCTION

Thank you for purchasing the Hu-Friedy SWERV Magnetostrictive Scaler. Caution: Federal law restricts this device to sale by or on the order of a licensed dental professional. The SWERV handpiece must be used with a Hu-Friedy SWERV control unit. This device generates ultrasonic vibrations intended for ultrasonic procedures: 1.) General Supra and Subgingival scaling applications; 2.) Periodontal debridement for non-surgical treatment of Periodontal diseases. Read this operation manual carefully before use and retain for reference.

SYMBOLS AND GENERAL SAFETY

Attention, consult accompanying documents

Risk of damage or harm

CE marking: Refers to directive 93/42 EEC, including EN 60601-1 and EN 60601-1-2

Dispose of this device and its accessories via methods approved for electronic devices and in compliance with the Directive 2002/96/CE

Medical equipment with respect to electrical shock, fire and mechanical hazards only in accordance with UL 60601-1, CAN/CSA C22.2 NO 601.1

Keep Dry

Handle With Care

SAFETY INSTRUCTIONS FOR HANDLING AND OPERATION

Read these instructions carefully and use only as intended or instructed. Safety instructions are intended to avoid potential hazards that could result in personal injury or damage to the device.


INSTALLATION AND ASSEMBLY

HANDPIECE CORD CONNECTION

Align the marks on the control unit and the handpiece cord plug. Firmly push the plug into the connector at the front of the control unit. (Fig. 1)

CAUTION:



1. Do not disconnect the handpiece cord when power to device is switched on and the footswitch is depressed.
2. Detach insert before removing handpiece cord.
3. Grip the plug of the handpiece cord when removing.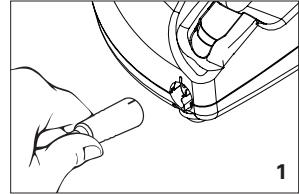

ULTRASONIC INSERT CONNECTION

Select appropriate kHz insert (25 or 30) for the generator. Fill handpiece with water. While holding the handpiece in an upright position, activate the purge function or the foot control to remove any air bubbles that may be trapped inside the handpiece. Lubricate the rubber o-ring on the sterilized insert with water prior to placing it into the handpiece. Fully seat the insert with a gentle push-twist motion. Do not force the insert into the handpiece.

⚠ WARNING: Always use water when operating the unit to keep the handpiece and insert cool. Minimum recommended setting >10 ml/min. Maximum recommended setting < 50 ml/min.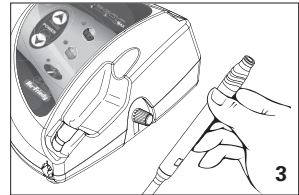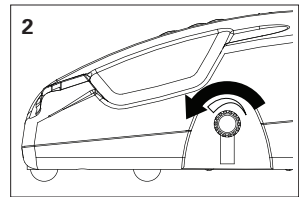

FOR OPERATION

- Verify that the water supply is adequate and clean prior to beginning each procedure.
- Purge handpiece.
- Use a Hu-Friedy disposable barrier sleeve (UMBS) on the handpiece. Replace the barrier between patients. Properly dispose of the barrier in accordance with guidelines for contaminated waste materials.
- Place insert in handpiece.
- Adjust the power level to the recommended mode.
- Depress the foot control to start insert vibration.
- Turn the water adjustment knob counterclockwise gradually to increase the water flow. (Fig. 2)

WHEN TREATMENT IS COMPLETED

1. Release the foot control and place the handpiece in the handpiece holder. (Fig. 3)
2. Turn off the power switch.
3. Hold handpiece in upright position. Gently pull the insert out of the handpiece opening. Ultrasonic inserts are easily interchangeable for various applications and procedures.
4. Close the dental unit's water valve.

CLEANING AND STERILIZATION

PROCEDURES FOR CLEANING HANDPIECE

1. Turn off the power switch.
2. Remove the insert from the handpiece.
3. Carefully wipe all surfaces of the handpiece and handpiece cord, which are contaminated by touch, contact with instruments or other inanimate objects during dental treatment with a soft cloth using only approved hospital grade surface disinfectants (EPA-registered, VAH / DGHM, RKI approval or CE marked).
4. Always follow the manufacturer's directions for disinfectant use.
5. The use of an abrasive wipe or cleaning solution will damage the surface finish of the control unit.
6. Phenols, iodophors, hypochlorite/household bleach and citric acid based disinfectants are not recommended.

NOTICE: The following websites provide additional information on safety and asepsis procedures:
<http://www.osap.org/indec.cfm> and <http://www.cdc.gov/OralHealth/infectioncontrol/index.htm>

CAUTION: Do not sterilize the handpiece or handpiece cord.

PROCEDURES FOR HU-FRIEDY ULTRASONIC INSERTS (sold separately)

1. Clean, inspect and steam sterilize before each use.
2. For complete instructions refer to cleaning and sterilization instructions provided with the ultrasonic insert.

SERVICE

Please send your SWERV handpiece with a copy of the invoice to your dealer or an authorized Hu-Friedy repair center. Shipping your product in its original packaging will offer the best protection against damage during shipping. If sending directly to an authorized Hu-Friedy repair center, please include your name and address, and the name of your dealer. Before shipment all items must be cleaned as described in the CLEANING AND STERILIZATION section of this User Manual.

TROUBLESHOOTING

See User Manual of your SWERV Control Unit for details or go to Hu-Friedy.com.

DISPOSAL AND RECYCLING


Dispose of this device and its accessories via methods approved for electronic devices and in compliance with the Directive 2002/96/CE. At the end of their life, ultrasonic inserts should be discarded in safety containers designed for this purpose.

HELPFUL INFORMATION

The use of higher than recommended power settings may cause excessive wear or tip breakage.

Always remember to properly seat the insert to the handpiece.

Scaling efficiency can diminish significantly with worn inserts (Fig. 4). Inferior performance and poor water delivery can result from worn, damaged, bent or altered stacks or working ends. For optimal scaling efficiency, we recommend checking inserts regularly for signs of wear.

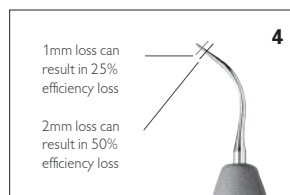CHECKING ULTRASONIC INSERTS FOR WEAR:

1. Locate the diagram of the insert to be measured on the wear guide (HF-466).
2. Rotate the insert until it is flat against the card.
3. Evaluate working end condition, comparing length to red/blue lines.

GUARANTEE – SWERV HANDPIECE ASSEMBLY

Hu-Friedy, at its sole option, will repair or replace for a period of one year following the date of the first purchase as new item, any SWERV Handpiece Assembly ("Handpiece") that fails as a result of a defect in materials or workmanship. Purchaser must return the defective Handpiece (postage prepaid) and must provide Hu-Friedy with the guarantee registration card or proof of purchase.

Excluded from this guarantee are damages in cases of:

- the Handpiece having been used outside of normal operating conditions;
- the Handpiece being damaged in transit;
- the Handpiece having been modified:
- non-compliance with the manufacturer's installation instructions (water pressure, mains voltage);
- use with an electrical system that does not comply with regulations in force;
- uses other than those specified in this manual;
- non-compliance with instructions contained in this manual;
- service to the Handpiece being provided by non-authorized persons;
- components exhibiting normal wear from usage;
- damages/defects being caused by force majeure or any other condition that is beyond the control of Hu-Friedy.

Repair service or replacement provided under the guarantee neither extends the guarantee period nor begins new guarantee period. THE EXPRESS WARRANTIES DESCRIBED ABOVE AND ANY APPLICABLE IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY AND FITNESS, ARE LIMITED AS STATED HEREIN. EXCEPT AS SET FORTH ABOVE, DAMAGES FROM BREACH OF SUCH WARRANTIES ARE LIMITED TO THE COST OR REPLACEMENT, AT HU-FRIEDY'S SOLE OPTION. CONSEQUENTIAL OR INCIDENTAL DAMAGES RESULTING FROM BREACH OF ANY APPLICABLE EXPRESS OR IMPLIED WARRANTIES ARE HEREBY EXCLUDED.

Please contact Hu-Friedy for instructions on obtaining warranty service.

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©2010 Hu-Friedy Mfg. Co., LLC. Hu-Friedy and the Hu-Friedy oval are trademarks of Hu-Friedy Mfg. Co., LLC.
Hu-Friedy Mfg. Co., LLC