Pantone 326C
Pantone 1945C
Pantone 425C
Black

6.5"

3"

DANGER

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For professional use only. Use with water at temperature between 90° and 140° F (32° to 60° C), demineralized water is ideal.

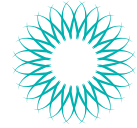
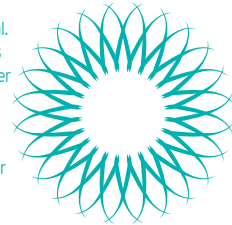
Presoak: Add 1 packet per 1 gallon (3.8L) of water. Soak for 3-5 minutes. Rinse thoroughly. **Linen Presoak:** Add 1 to 2 packets per 1 gallon (3.8L) of water. Soak clothing for 10 minutes. Proceed with normal washing. **Ultrasonic Cleaning:** Add 1 packet per 1 gallon (3.8L) of water. Rinse instruments thoroughly, dry and proceed with sterilization. **Evacuators:** Add 2 packets per 1 gallon (3.8L) of water. Insert evacuator hose, turn system on full open for one minute. Suggested maintenance twice weekly.

Store in a cool, dry place. Change daily or when visibly soiled.


Contains: Protease enzyme, cellulase enzyme, and d-limonene. For more ingredient information, please visit Hu-Friedy.com or call 1-800-Hu-Friedy.

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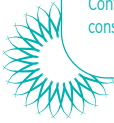3"

HINGE AREA

Avoid breathing dust/fume/gas/mist/vapours/spray. Wash hands, forearms and face thoroughly after handling. Wear protective gloves/protective clothing/eye protection/face protection. In case of inadequate ventilation wear respiratory protection. **IF ON SKIN:** Wash with plenty of water. Take off contaminated clothing and wash it before reuse. If skin irritation occurs: Get medical advice/attention. **IF INHALED:** Remove person to fresh air and keep comfortable for breathing. If experiencing respiratory symptoms: Call a POISON CENTER/doctor. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/doctor. Dispose of contents/container in accordance with local, regional, national and/or international regulations.

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