


ORTHODONTIC

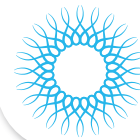

ORTHO LINGUAL COLLECTION


REDEFINING THE ART
OF ORTHODONTICS

How the best perform




Hu-Friedy's

ORTHO LINGUAL COLLECTION

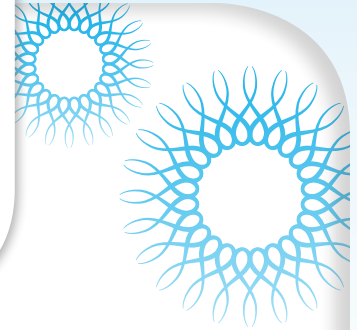

14% TO 29% THINNER
ORBIT FORMED JOINT
THAN STANDARD

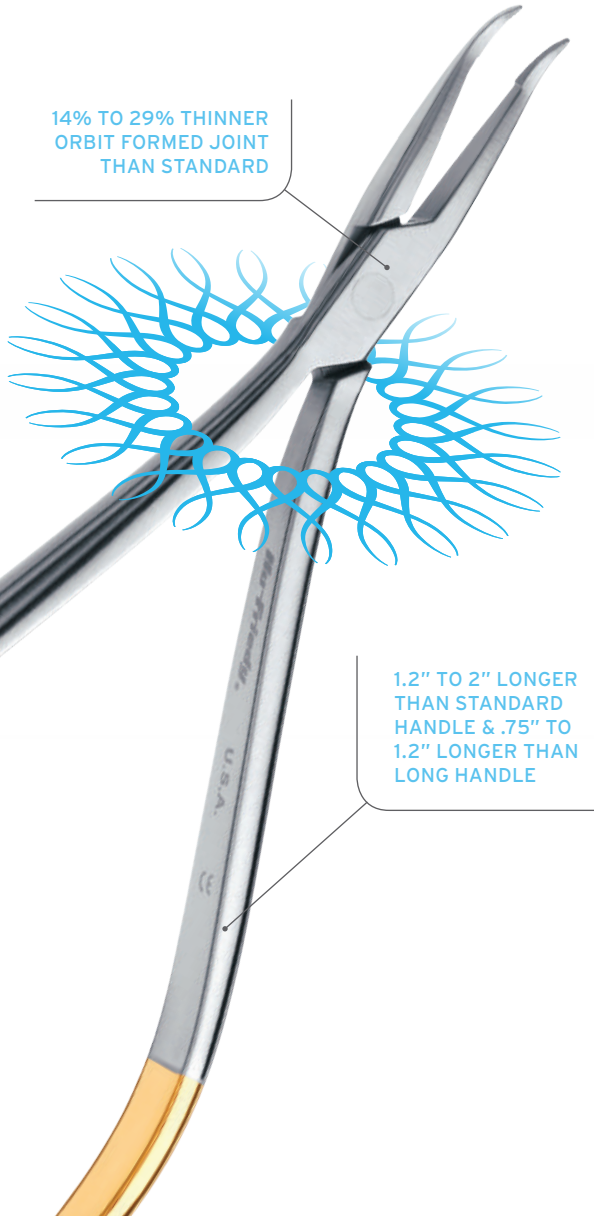

1.2" TO 2" LONGER
THAN STANDARD
HANDLE & .75" TO
1.2" LONGER THAN
LONG HANDLE

The Ortho Lingual Collection consists of ten instruments that feature:

- Optimal **intraoral access**
- Ideal **visibility** for clinician
- Enhanced **comfort** for patient
- More **time savings** allowing for additional patient education

Every ortho instrument from Hu-Friedy is created using the highest quality materials, processes and quality assurance.

DID YOU KNOW?

Hu-Friedy Orthodontic instruments are designed to withstand the rigors of steam sterilization while maintaining their beauty and functionality.


LINGUAL CUTTERS



Lingual Distal End Cutter

| 678-700

Wire size: .012" to .018"

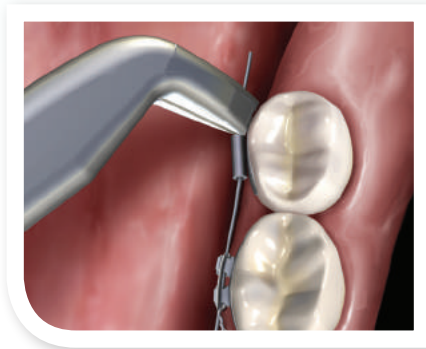


Occlusal view of the Lingual Distal End Cutter cutting the excess archwire distal to the buccal tube.



Lingual Pin & Lig Cutter

| 678-701

Wire size: < = .012"

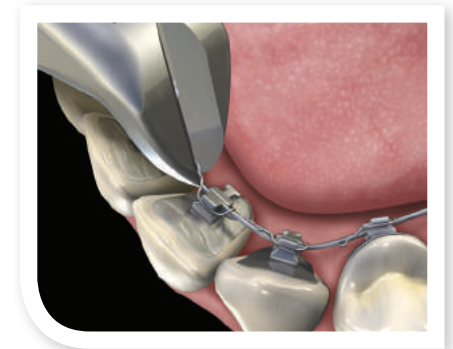

Occlusal view of the Lingual Pin & Lig Cutter cutting the excess steel ligature.

LINGUAL PLIERS



Lingual Weingart Pliers

| 678-702

Wire size: $\leq .030$ "



Occlusal view of the Lingual Weingart Pliers guiding the archwire into the bracket slot.



Lingual Weingart Pliers,
90° Bend

| 678-702-90

Wire size: $\leq .030$ "



LINGUAL PLIERS




Lingual Mathieu Pliers
| 678-705



Occlusal view of the Lingual Mathieu Pliers tying steel ligature to hold the arch wire into bracket slot.



Lingual Bracket
Removing Pliers
| 678-703

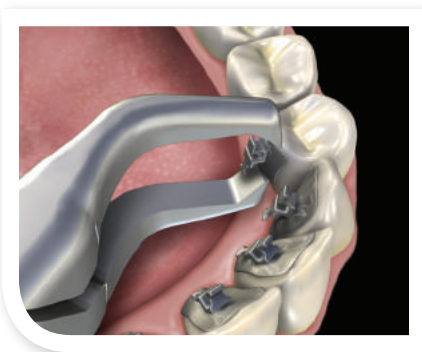

Lingual view of the Lingual Bracket Removing Pliers debonding the bracket.


LINGUAL PLIERS



Lingual
Hammerhead®
NiTi Pliers

| 678-704

Wire size: $\leq .025''$

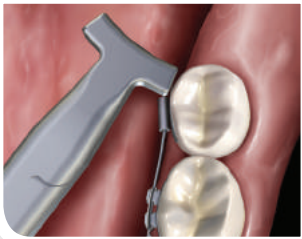


Occlusal view of the Lingual
Hammerhead NiTi Pliers bending NiTi
wire distal to the buccal tube.



Lingual
Hammerhead
NiTi Pliers with
Serrations

| 678-704SER

Wire size: $\leq .025''$

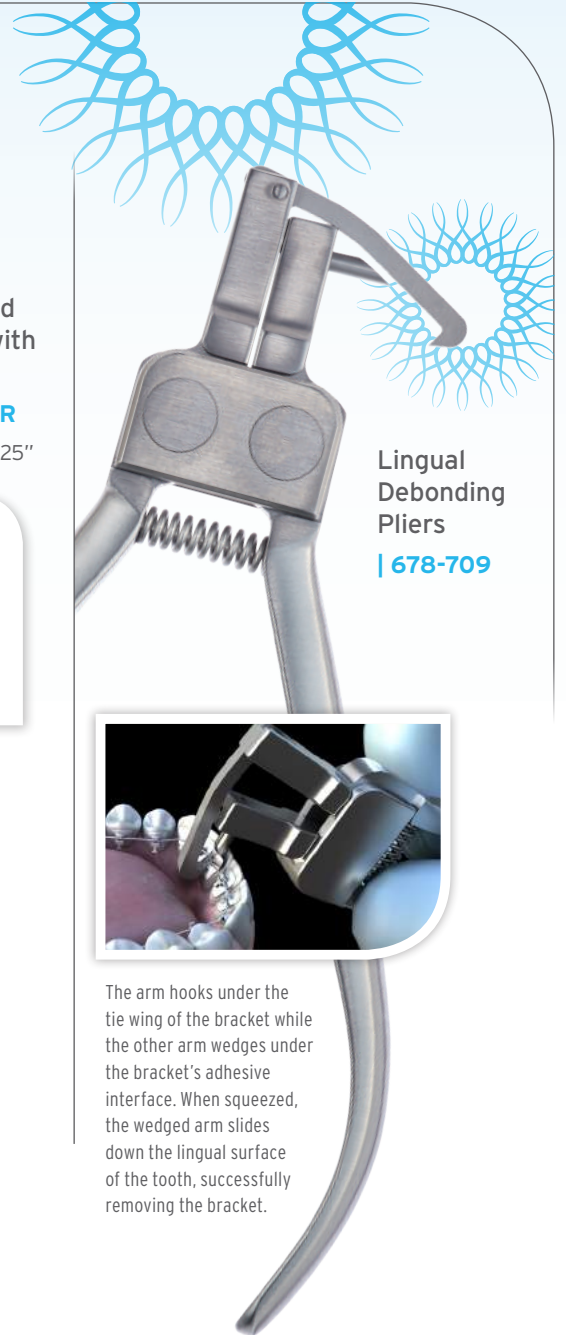


Lingual
Debonding
Pliers

| 678-709



The arm hooks under the
tie wing of the bracket while
the other arm wedges under
the bracket's adhesive
interface. When squeezed,
the wedged arm slides
down the lingual surface
of the tooth, successfully
removing the bracket.


LINGUAL HAND INSTRUMENTS

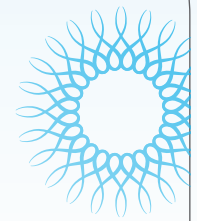

Lingual Ligature Director

| 678-706

Wire size: $\leq .021'' \times .025''$

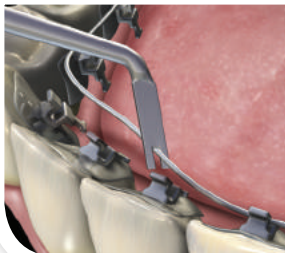

Ligature Side: Occlusal view of the Lingual Ligature Director guiding the archwire into the bracket slot.




Picker Side: Occlusal view of the Lingual Ligature Director tucking the excess steel ligature underneath the bracket.



Lingual Ligature Director, U Shape

| 678-707

Wire size: $\leq .021'' \times .025''$

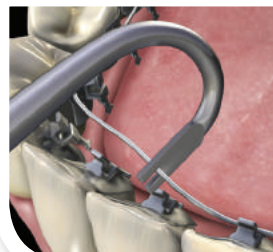


Occlusal view of the U Shaped Lingual Ligature Director guiding the archwire into the bracket slot.




Lingual Torquing Key with Picker

| 678-708

Slot size .018''

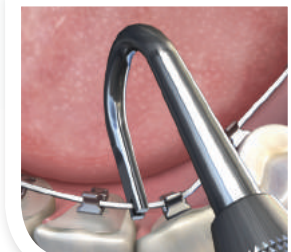

Occlusal view of the Torquing Key guiding the archwire into the bracket slot.

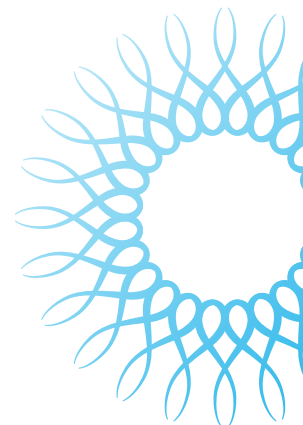

RECYCLE BROKEN & OLD INSTRUMENTS

HU-FRIEDY
PROGRAMS



When your instrument blades cannot be sharpened anymore, take advantage of Hu-Friedy's Envirodent® Instrument Recycling Program. Your old, broken instruments will be recycled to help create new car bumpers, bridges and street lights. For every 12 you recycle, we'll give you 1 FREE Hu-Friedy instrument up to \$47!



To start recycling visit
Envirodent.com

THE HU-FRIEDY ORTHO LINGUAL COLLECTION

CUTTERS

| | |
|---------------------------|---------|
| Lingual Distal End Cutter | 678-700 |
| Lingual Pin & Lig Cutter | 678-701 |

PLIERS

| | |
|--|------------|
| Lingual Weingart Pliers | 678-702 |
| Lingual Weingart Pliers, 90° | 678-702-90 |
| Lingual Bracket Removing Pliers | 678-703 |
| Lingual Hammerhead NiTi Pliers | 678-704 |
| Lingual Hammerhead NiTi Pliers with Serrations | 678-704SER |
| Lingual Mathieu Pliers | 678-705 |
| Lingual Debonding Pliers | 678-709 |

HAND INSTRUMENTS

| | |
|------------------------------------|---------|
| Lingual Ligature Director | 678-706 |
| Lingual Ligature Director, U Shape | 678-707 |
| Lingual Torquing Key with Picker | 678-708 |

For more information about
Hu-Friedy's Total Solutions visit
HU-FRIEDY.COM

PERIODONTAL



RESTORATIVE



ORTHODONTIC




INSTRUMENT
MANAGEMENT



ENDODONTIC



SURGICAL

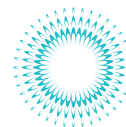

ULTRASONIC
SCALING



Hu-Friedy Mfg. Co., LLC: 3232 N. Rockwell St. | Chicago, IL 60618 | USA
Telephone: 1-800-Hu-Friedy (1-773-975-6100) | Website: Hu-Friedy.com

©2017 Hu-Friedy Mfg. Co., LLC. All rights reserved. [289] HF-899/1117

Hu-Friedy