



# CompoSculp

| CASE STUDY BY DR. DIDIER DIETSCHI

## ANTERIOR PROCEDURE



FIGURE 1

### PREOPERATIVE VIEW

Enamel opacities will be corrected and anterior teeth form enhanced.



FIGURE 2

Rubber dam is placed to achieve proper isolation. Restorative site receiving treatment is prepped.



FIGURE 3

Composite material is placed inside the buccal cavity with the paddle end of PFIDD5/6.

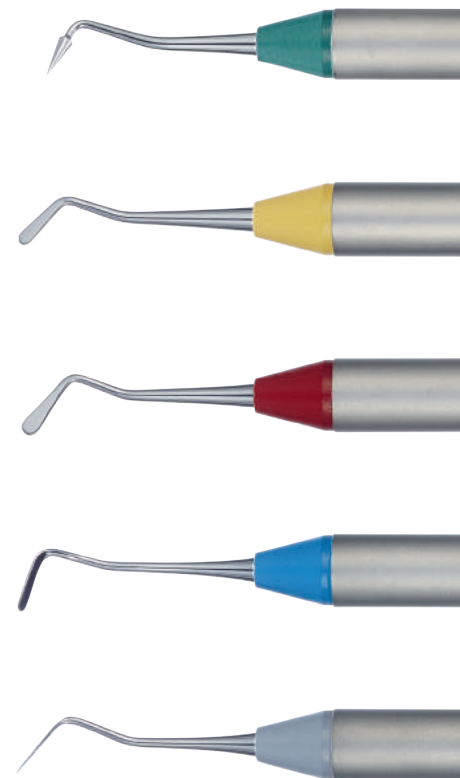


FIGURE 4

Composite material is applied and spread using the fine blades of the PFIDD9/10.

### Excellence and Innovation in Composite Restorations

CompoSculp instruments were designed by Dr. Didier Dietschi with the intention of aiding clinicians in achieving superior esthetic restorations. The CompoSculp kit is comprised of 5 complimentary composite instruments for both anterior and posterior restorations.



How the best perform

**Hu-Friedy**



FIGURE 5



FIGURE 6

Shaping and contouring the composite to modify proximal tooth contours is achieved by use of the off-angle blades on the PFIDD7/8.



FIGURE 7

After finishing and polishing, the restoration is complete.



Didier Dietschi, D.M.D., PhD, Privat-Dozent  
Specialist SVPR  
Associate Professor

Dr. Didier Dietschi was licensed in 1984 and got his doctoral and Privat Dozent degrees in 1988 and 2004, respectively, at the University of Geneva, Switzerland. He also got a PhD degree in 2003 at the University of ACTA, Netherlands. Following a 6-year period of full time teaching and research activity in Operative Dentistry and Periodontology, he started a part time activity in a private office in Geneva, dedicated to aesthetic restorative dentistry. He holds now positions of adjunct Professor at CASE Western University (USA) and senior lecturer at the University of Geneva.

Dr. Dietschi has published more than 95 clinical and scientific papers and book chapters on adhesive and aesthetic restorations; he also co-authored the book "Adhesive Metal-free Restorations", edited in 1997 by Quintessence and translated in 7 languages. Dr. Dietschi is lecturing internationally on adhesive and aesthetic restorations.

For more information regarding Dr. Didier Dietschi's teaching agenda, please visit:  
[www.edudentinternational.com](http://www.edudentinternational.com)

## COMPOSCULP ORDERING INFORMATION



Part Code	Grip Color	Smooth Handle Description
PFIDD1/2	Green	Dietschi Composite #1/2
PFIDD3/4	Yellow	Dietschi Composite #3/4
PFIDD5/6	Red	Dietschi Composite #5/6
PFIDD7/8	Blue	Dietschi Composite #7/8
PFIDD9/10	Grey	Dietschi Composite #9/10
PFIDDCASS		Dietschi Composite Kit, Cassette*
PFIDDDIN		Dietschi Composite Kit, DIN Cassette**

Part Code	Grip Color	ResinEight Handle Description
PFIDD1/28	Green	Dietschi Composite #1/2
PFIDD3/48	Yellow	Dietschi Composite #3/4
PFIDD5/68	Red	Dietschi Composite #5/6
PFIDD7/88	Blue	Dietschi Composite #7/8
PFIDD9/108	Grey	Dietschi Composite #9/10
PFIDDCASS8		Dietschi Composite Kit, Cassette <sup>o</sup>
PFIDDDIN8		Dietschi Composite Kit, Cassette <sup>oo</sup>

\*Includes: PFIDD1/2, PFIDD3/4, PFIDD5/6, PFIDD7/8, PFIDD9/10 and IM6053 (5 inst. cass.)

\*\*Includes: PFIDD1/2, PFIDD3/4, PFIDD5/6, PFIDD7/8, PFIDD9/10 and IME14DIN53 (5 inst. DIN cassette, for European customers)

<sup>o</sup>Includes: PFIDD1/28, PFIDD3/48, PFIDD5/68, PFIDD7/88, PFIDD9/108 and IM6053 (5 inst. cass.)

<sup>oo</sup>Includes: PFIDD1/28, PFIDD3/48, PFIDD5/68, PFIDD7/88, PFIDD9/108 and IME14DIN53 (5 inst. DIN cassette, for European customers)

Learn more about CompoSculp and Dr. Dietschi at:  
[HU-FRIEDY.COM/CompoSculp](http://HU-FRIEDY.COM/CompoSculp)

©2016 Hu-Friedy Mfg. Co., LLC. All rights reserved. N2350/1216

